

# Oxysept Hydrogen Peroxide Disinfecting Solution



**Brand:**  
**Product Code:** 93359ED  
**Availability:** 2  
**Weight:** 908.00g  
**Dimensions:** 180.00mm x 65.00mm x 175.00mm

**Price: \$41.50**

## Short Description

Powerful disinfecting solution for contact lenses.

## Description

**The Oxysept<sup>®</sup> Disinfecting Solution/Neutralizer delivers powerful disinfection efficacy against a broad range of microorganisms<sup>1</sup>**

- Excellent kill rates provide patients with a high level of protection against potential microbial infection.
- Hydrogen peroxide is a highly effective lens care solution at killing ocular pathogens, including Acanthamoeba.<sup>2</sup>
- Potent bactericidal and antifungal activity meets all stand-alone disinfection efficacy criteria.<sup>1</sup>
- Superior disinfection efficacy results from prolonged exposure to 3% hydrogen peroxide before neutralization begins.
- Delayed neutralization ensures disinfection efficacy superior to the competition

## Directions

Always read the manufacturers instructions.

## **Daily clean**

Clean each of your lenses with a daily cleaner and rinse with a sterile saline solution before placing them in the appropriate basket of the lens holder.

## **Disinfect**

Disinfect with Oxysept<sup>®</sup> Disinfecting Solution. Fill the Oxysept<sup>®</sup> Disinfecting Solution/Neutralizer Cup to the fill line with solution.

## **Neutralize**

Neutralize with Oxysept<sup>®</sup> Neutralizing Tablets. Place one neutralizing tablet into the solution before inserting the lens holder and tightening the lid. Invert the Oxysept<sup>®</sup> Disinfecting Solution/Neutralizer Cup 3 consecutive times to wet the upper surface of the cup and the inside of the cap to ensure full neutralization. The Oxysept<sup>®</sup> Neutralizing Tablet will color the solution pink reminding you that you have added the neutralizing tablet at the beginning of the six (6) hour soak.

## **Soak**

Soak your contact lenses for a minimum of six (6) hours from the time you dropped the neutralizing tablet into the Oxysept<sup>®</sup> Cup. Your disinfected contact lenses are now ready to wear.

## **Once a week, remove protein buildup**

Add one Ultrazyme<sup>®</sup> Enzymatic Cleaner Tablet at the same time you add the Oxysept<sup>®</sup> Neutralizing Tablet to the disinfecting solution. Remove lenses and rub and rinse with a sterile saline solution prior to wearing your lenses.